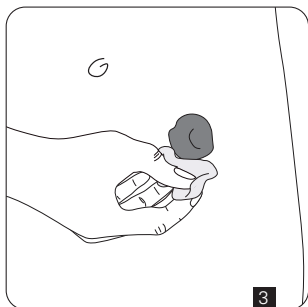
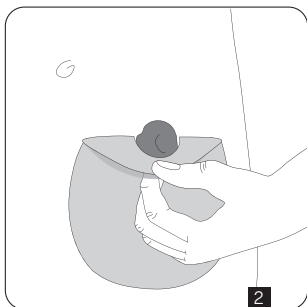
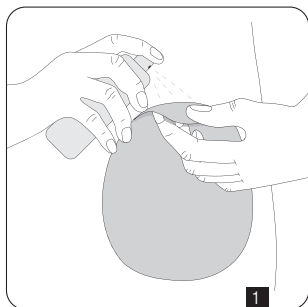
	Pročitajte upute za uporabu / Pročitajte uputstvo za upotrebu / Glejte navodila za uporabo / Consult instructions for use / Gebrauchsanweisung / Consultar las instrucciones de uso / Συμβουλευτείτε τις οδηγίες Χρήσης / Consultare le istruzioni per l'uso		Držati dalje od sunčeve svjetlosti / Držati dalje od sunčeve svetlosti / Hranite stran od sončne svetlobe / Keep away from sunlight / Vor Sonnenlicht schützen / No exponer a la luz solar / Διατηρείτε μακριά από το ηλιακό φως / Tenere al riparo dalla luce solare
	Jedinstvena identifikacija proizvoda / Jedinstvena identifikacija proizvoda / Edinstveni identifikator pripomočka / Unique device identifier / Einmalige Produktkennung / Identificador único del producto / Μοναδικό αναγνωριστικό τεχνολογικού προϊόντος / Identificazione unica del dispositivo		Internetska stranica s informacijama za pacijente / Internetska stranica sa informacijama za pacijente / Spletna stran z informacijami za pacijenta / Patient information website / Website mit Informationen für Patienten / Sitio web de información para pacientes / Ιστοσελίδα πληροφοριών για τον ασθενή / Sito web con le informazioni per i pazienti
	Oznaka CE / CE oznaka / Oznaka CE / CE mark / CE-Kennzeichnung / Marca-do CE / Σήμανση CE / Marcatura CE		Broj serije / Broj serije / Številka serije / Batch code / Chargenbezeichnung / Código de lote / Κωδικός παρτίδας / Codice del lotto
	Kataloški broj / Kataloški broj / Kataloška številka / Catalogue number / Bestellnummer / Número de catálogo / Αριθμός καταλόγου / Numero di catalogo		Čuvati suhim / Čuvati na suvom / Hranite na suhem / Keep dry / Vor Nässe schützen / Mantener seco / Διατηρείτε στεγνό / Tenere al riparo dall'umidità
	Medicinski proizvod / Medicinski proizvod / Medicinski pripomoček / Medical device / Medizinprodukt / Producto sanitario / Ιατροτεχνολογικό προϊόν / Dispositivo medico		Datum proizvodnje / Datum proizvodnje / Datum proizvodnje / Date of manufacture / Herstellungsdatum / Fecha de fabricación / Ημερομηνία κατασκευής / Data di fabbricazione
	Zapaljivo / Zapaljivo / Vnetljivo / Flammable / Entzündlich / Inflamable / Εύφλεκτος / Infiammabile		Rok uporabe / Rok upotrebe / Uporabno do / Use-by date / Verfallsdatum / Fecha de caducidad / Ημερομηνία λήξης / Data di scadenza
	Proizvođač / Proizvođač / Proizvajalec / Manufacturer / Hersteller / Fabricante / Κατασκευαστής / Fabricante		Distributer / Distributer / Distributer / Distributor / Verteiler / Distribuidor / Διανομέας / Distributore

## HR

### Upute za uporabu

**OPIS: Lentell O'Spray.** Sprej za uklanjanje ljepila. Pomagalo za stome

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**EN**

## Instructions for use

**DESCRIPTION: Lentell O'Spray.** Adhesive remover spray. Ostomy appliance

**INTENDED PURPOSE:** Reduces the adhesiveness of ostomy appliances and removes adhesive residues from the skin of colostomy, ileostomy and urostomy patients. Be sure to strictly follow the instructions for use and to contact your stoma nurse or doctor if necessary.

**CAUTION:**

- Rely on the guidance of medical personnel while using the product. Our company does not bear any responsibility for indirect damage or losses caused by the use of product that does not comply with the instructions for use.
- Apply directly to the skin around the stoma. Use on intact skin only. Avoid delicate or sensitive areas of skin (such as eyes, mouth, ears, nose, vagina).
- Patch test a small area prior to use. Discontinue the use if irritation or allergies occur.
- Close the spray opening immediately after use to avoid contamination.
- Avoid direct inhalation.
- If during the use of this device or as a result of its use a serious incident occurs, please report it to the manufacturer and to your national authority.
- Do not use the product after the expiration date.

**CONTRAINDICATIONS:**

- This product is not sterile, and it should not be used by people with a weakened immune system, e.g. patients suffering from leukaemia, whose immune system is weakened by drugs.
- The product should not be used on premature babies and new-born babies in incubators.

**WARNINGS:** Pressurised container - protect from sunlight and do not expose to temperatures exceeding 50°C.

Do not pierce or burn, even after use. Do not smoke nearby.

**STORAGE:** Store the product in a cool and dry environment. Keep away from sources of ignition.

Keep out of reach of children.

**DISPOSAL:** After use dispose of the product according to national regulations.

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The Lentismed logo is a registered trademark of Lentismed d.o.o. TM indicates trademarks of Lentismed d.o.o.