Lentell[™]

Upotreba
konveksnih
pomagala u
brizi za stome

Jedan od najvažnijih aspekata brige za stome je osiguravanje da je pomagalo sigurno i dobro pričvršćeno¹. Pokazano se da je 75 % slučajeva iritacije peristomalne kože uzrokovano curenjem sadržaja ispod sustava za stome. Pritom je zabrinjavajuće da oko 40 % bolesnika sa stomom ne prepozna da ima problem s iritacijom peristomalne kože, a više od 80 % onih s dijagnozom bolesti kože ne potraži stručnu pomoć.

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1 Colwell J. Selection of a pouching system. Wound, Ostomy and Continence Nurses Society Core Curriculum: Ostomy Management. Philadelphia, PA: Wolters Kluwer; 2016:120–130.

2 Herlufsen P, Olsen AG, Carlsen B, Nybaek H, Jemec GB, Karlsmark T, Laursen TN. Study of peristomal skin disorders in patients with permanent stomas. *British Journal of Nursing*. 2026; 15(16): 854–62.

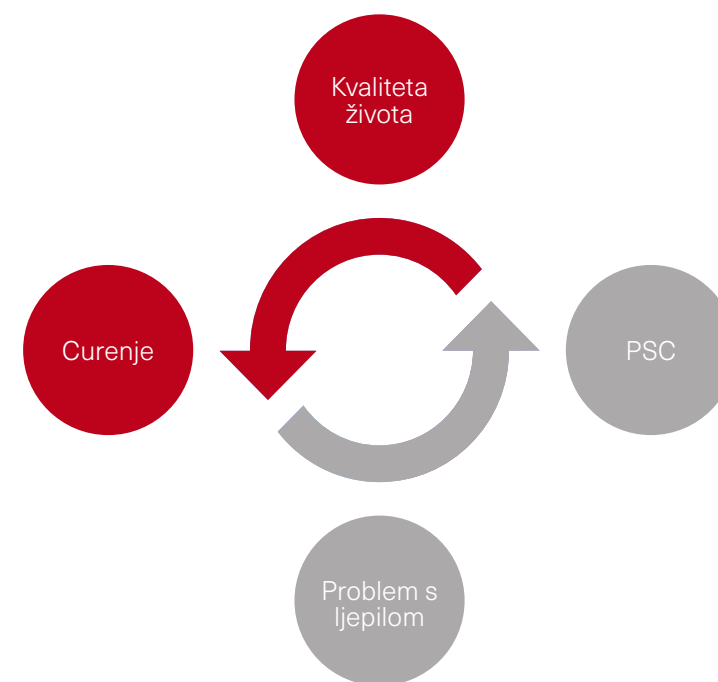
3 Rolstad BS, Erwin-Toth PL. Peristomal skin complications: prevention and management. *Ostomy Wound Manage*. 2004; 50(1):68–77.

4 Hoeflok J, Salvadalena G, Pridham S, Droste W, McNichol L, Gray M. Use of Convexity in Ostomy Care: Results of an International Consensus Meeting. *J Wound Ostomy Continence Nurs*. 2017;44(1): 55-62.

5 Hoeflok J, Kittscha J, Purnell P. Use of Convexity in Pouching. *Journal of Wound, Ostomy and Continence Nursing*. 2013; 40(5): 506–12.

6 Rat P, Robert N, Fernandes I, Edmond D, Mauvais F. Evaluation of a one-piece soft convex ostomy appliance: a prospective, multicentre, open-label pilot study. *British Journal of Nursing*. 2018; 27(16): S20–S28.

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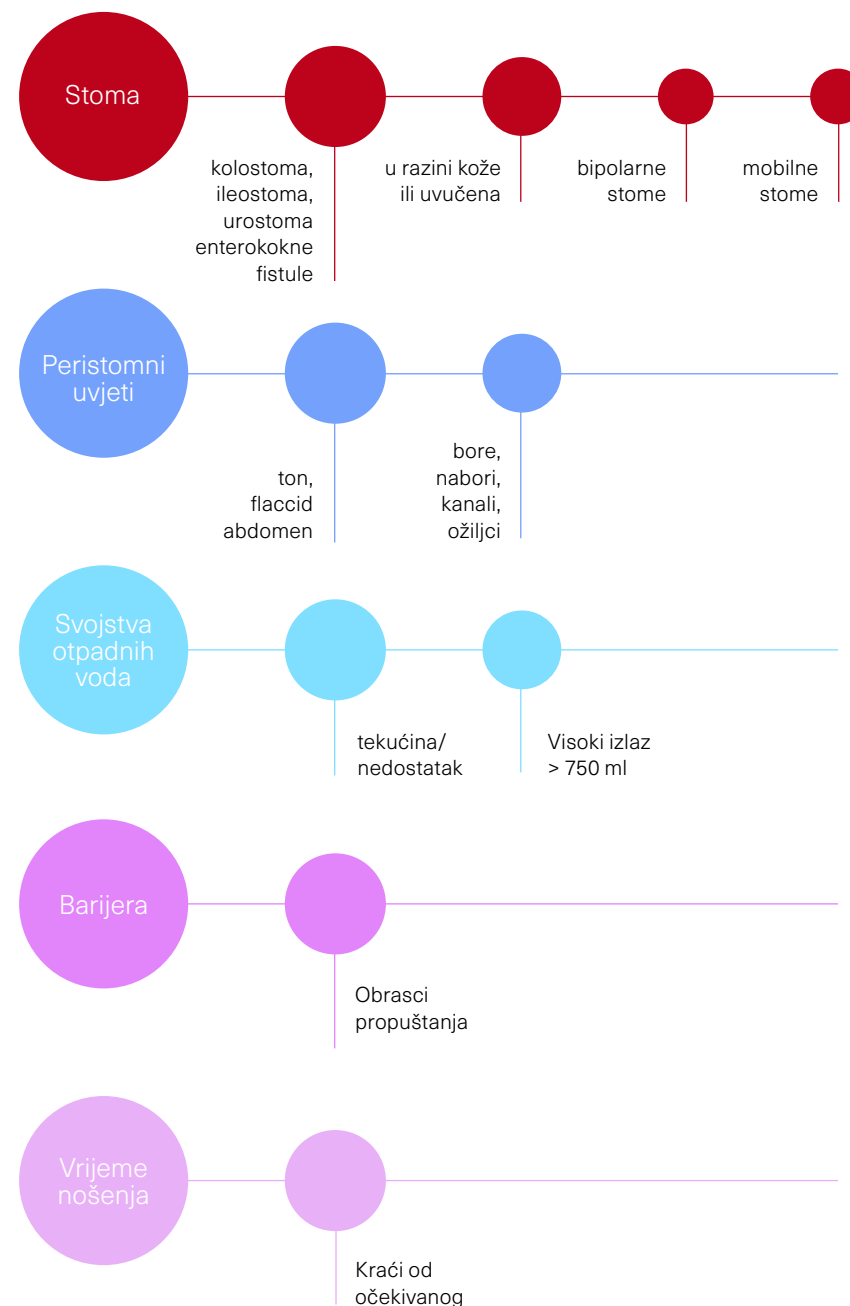
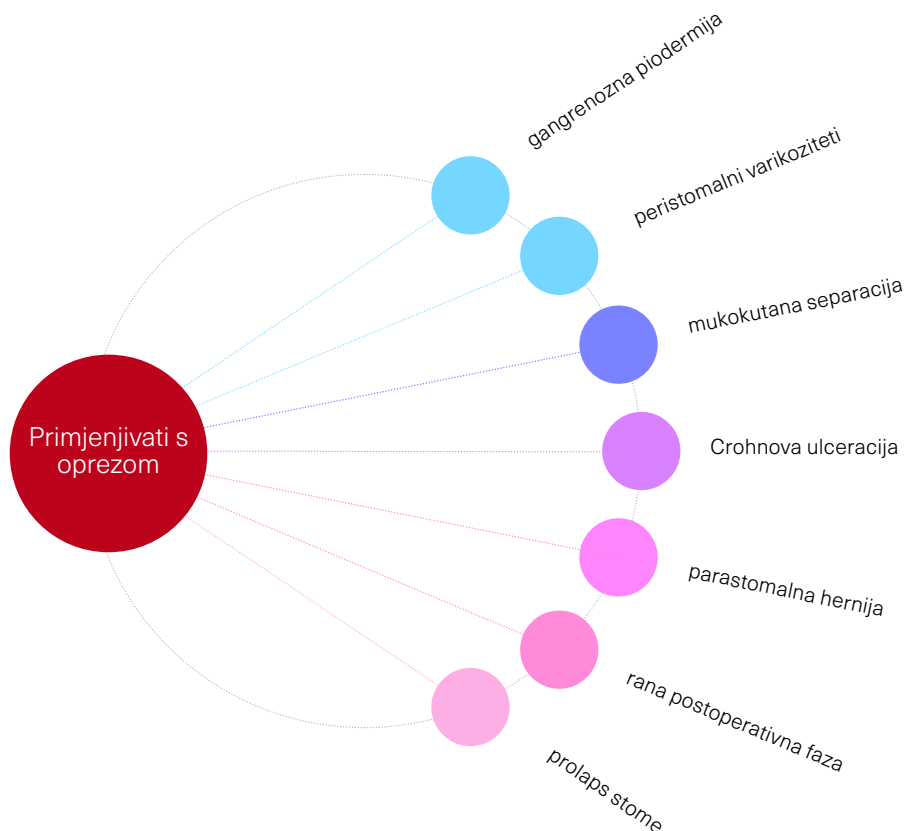
8 Boyd K, Thompson MJ, Boyd-Carson W, Trainor B. Use of convex appliances. *Nursing Standard*. 2004; 18(20): 37–38.

9 Erwin-Toth P. Ostomy Pearls. *Advances in Skin & Wound Care*. 2003 16(3): 146–52.

10 Wound, Ostomy and Continence Society. WOCN Society position paper: Avoidable versus unavoidable pressure ulcers (injuries). 2017. https://cdn.ymaws.com/member.wocn.org/resource/resmgr/document_library/stoma_complications_best_pra.pdf

11 Part 2: Assessment and Management of Stomal Complications: A Framework for Clinical Decision Making. 2004. <https://www.hmpgloballearningnetwork.com/site/wmp/article/part-2-assessment-and-management-stomal-complications-framework-clinical-decision-making>

Procjena peristomalnog integriteta kože je još jedan važan faktor prilikom odlučivanja hoće li se upotrijebiti konveksna pločica. Potrebno je napraviti potpuna analiza stanja kože, uključujući oštećenja peristomalne kože.[5]

Konveksnost treba koristiti s oprezom jer pritisak koji se vrši može pogoršati postojeće uvjete, kao što su gangrenozna piodermija, peristomalni varikoziteti, mukokutana separacija, Crohnova ulceracija, parastomalna hernija, prolaps stome, ponajviše u ranoj postoperativnoj fazi.[8]

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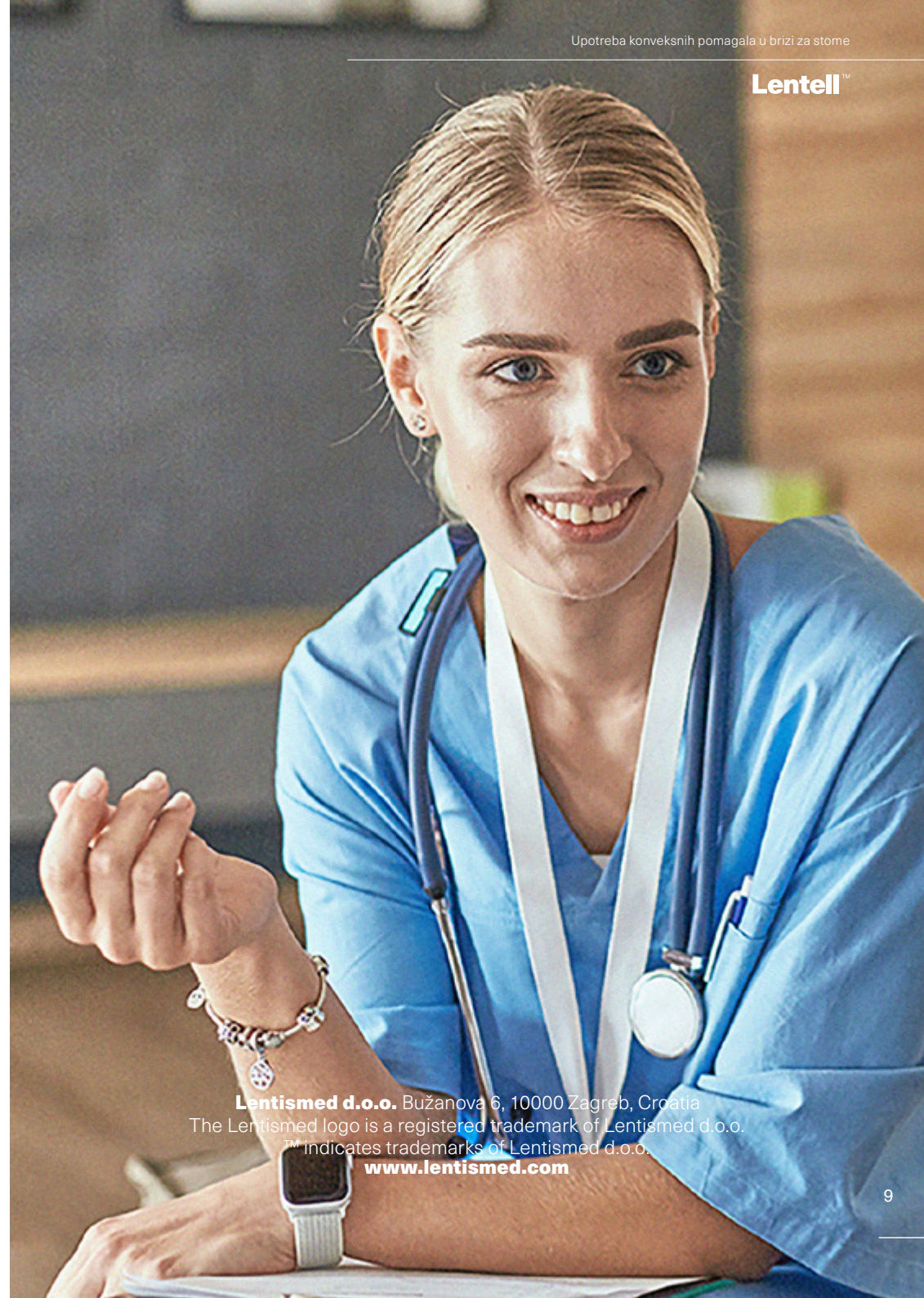
Lentimed d.o.o. Bužanova 6, 10000 Zagreb, Croatia
The Lentimed logo is a registered trademark of Lentimed d.o.o.
™ indicates trademarks of Lentimed d.o.o.
www.lentimed.com